
AURA® & AURA HOME®

Special needs strollers

me



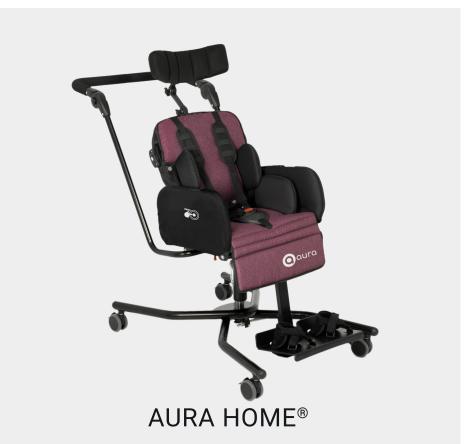

This is a medical device.

Use it in accordance with the user manual or the label.

Description **AURA**®

Indications

AURA® is a special needs stroller dedicated to children who are unable to move independently due to locomotor dysfunctions. It is equipped with numerous useful solutions which make it easier to take care of the user by providing safety and comfort. Adjustable backrest height, seat depth and footrest length make the stroller grow together with the child. An advanced trunk and hip support system provides proper lateral stabilization. In combination with 5-point safety belts, it significantly affects the maintenance of the correct sitting position. The backrest is equipped with a headrest that can be easily adjusted to the shape of the user's head, protecting it from falling to the side. The tilt-in-space function, along with the reclining backrest and adjustable footrest angle, allow you to put the stroller in the reclining position easily and quickly. This is very useful when you need to change the position of the user from sitting to lying down, depending on the user's current state of health and well-being. The reversible seat can be put in the forward or rearward position. The child can explore the surroundings while sitting in the stroller or, if necessary, have direct eye contact with the caregiver, which increases their sense of security.

AURA® is ideal for long walks. PU wheels and rear shock absorbers effectively reduce vibrations while riding. Central rear wheel brake ensures safety when stopping the stroller, as well as when putting the child in and out of the seat. Front wheels equipped with a directional locking system, make it easier to drive the stroller in more difficult terrain, such as on grass or forest paths. The adjustable push handle allows you to adjust its height to fit the height of the caregiver. The design of the stroller and removable wheels, allow it to be folded to a compact size, so it will fit into small trunks. The stroller is made of high-quality materials that guarantee durability and reliability.

- Cerebral palsy various types and forms.
- Brain and spinal cord injuries with lower limbs paralysis and weakness, balance and motor coordination disorders with different etiology.
- Demyelinating disease (e.g. multiple sclerosis SM).
- Meningocele.
- Muscles diseases with weakness and paralysis (dystrophy, myasthenia gravis, myositis).
- · Genetic disorders.
- Degeneration and neurological diseases.
- Other condition connected with muscles weakness, paralysis and musculoskeletal disorders.
- · After head and spinal cord injuries and spine trauma with paralysis and paresis.



Special features



Stable structure

The device ensures adequate stability and safety of user.



Easy to use

Intuitive way of adjustment and easy assembly.



Lightweight construction

The product is made of light metal alloys which positively affects the low weight of the device.



Reflective elements

The product has reflective elements to ensure your safety.



Tilt-In-Space

The device is able to change hip angle.



Movable seat

You can turn the seat easily so that the child may be facing towards caregiver or movement direction.



24-month warranty

We offer 24-month warranty on the product. We also provide after sales service.

Colors


Waterproof upholstery **HYDRO™**







Independent adjustment of chest and pelvic supports

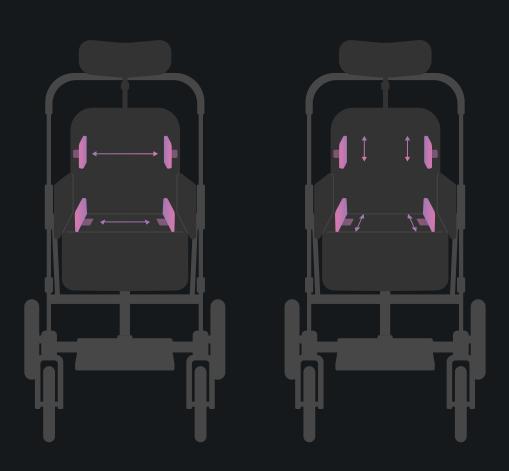

You have full control

Front swivel wheels with directional locking system make it easy to maneuver the stroller. Rear wheels equipped with shock absorbers effectively reduce vibrations while riding.



Frame change

Changing the frame has never been so easy. In just a few steps, you can change the frame from an outdoor to a HOME version. Additionally, while using the outdoor frame, you can mount the seat in the forward or backward position.



Compact size

The stroller is easy to fold thanks to its construction and removable wheels, and the compact dimensions make it fit even into small car trunks.



You can easily change the frame of the stroller for the HOME version.



Tilt-In-Space

You can smoothly change the angle of the whole seat. This function is important for better pressure distribution on sensitive body areas (antibedsore prophylaxis).



Short backrest

With HexagonPro[™] headrest.



Long backrest

With adjustable head stabilizers.



Adjustable armrests


You can effortlessly adjust the height of the armrests at any time, even when the user is sitting in a stroller.



Standard equipment



Important measurements

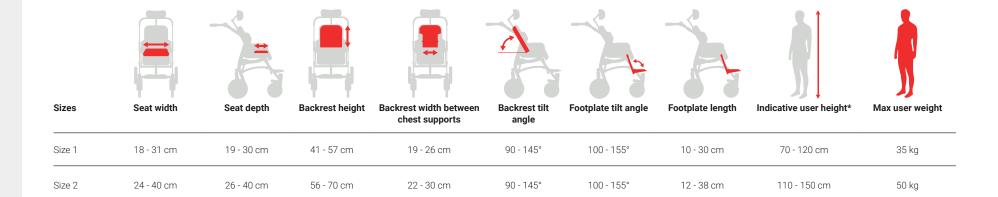

Weight and dimensions

Size	Width	Width after folding	Length	Length after folding	Height	Height after folding	Weight
Size 1	60 cm	60 cm	104 cm	91 cm	80 - 120 cm	50 cm	20 kg
Size 2	70 cm	70 cm	105 cm	93 cm	80 - 125 cm	60 cm	24 kg



Scan the QR code to see the full table.

^{*}Height cannot be the main parameter while choosing the product size.

Note: In extreme positions of the handle, it may collide with the backrest when changing its angle. In such cases, the handle should be raised while adjusting.

Note: Measurement tolerance range is +/- 1 cm / ½ in.

Note: Approximate values, depending on the product configuration.

Description AURA HOME®

AURA HOME® is an indoor stroller that is an ideal solution during the daily rehabilitation of people with limited locomotor abilities. It is equipped with numerous useful solutions which make it easier to take care of the user by providing safety and comfort.

Adjustable backrest height, seat depth and footrest length make the stroller grow together with the child. An advanced trunk and hip support system provides proper lateral stabilization. In combination with 5-point safety belts, it significantly affects the maintenance of the correct sitting position. The backrest is equipped with a headrest that can be easily adjusted to the shape of the user's head, protecting it from falling to the side.

The tilt-in-space function, along with the reclining backrest and adjustable footrest angle, allow you to put the stroller in the reclining position easily and quickly. This is very useful when you need to change the position of the user from sitting to lying down, depending on the user's current state of health and well-being.

The adjustable push handle can be easily adjusted to the height of the caregiver. The frame of the stroller is equipped with wheels with brakes.

Indications

- Cerebral palsy various types and forms.
- Brain and spinal cord injuries with lower limbs paralysis and weakness, balance and motor coordination disorders with different etiology.
- Demyelinating disease (e.g. multiple sclerosis SM).
- · Meningocele.
- Muscles diseases with weakness and paralysis (dystrophy, myasthenia gravis, myositis).
- · Genetic disorders.
- Degeneration and neurological diseases.
- Other condition connected with muscles weakness, paralysis and musculoskeletal disorders.
- · After head and spinal cord injuries and spine trauma with paralysis and paresis.



Special features



Stable structure

The device ensures adequate stability and safety of user.



Easy to use

Intuitive way of adjustment and easy assembly.



Lightweight construction

The product is made of light metal alloys which positively affects the low weight of the device.



Reflective elements

The product has reflective elements to ensure your safety.



Tilt-In-Space

The device is able to change hip angle.



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
Movable seat

You can turn the seat easily so that the child may be facing towards caregiver or movement direction.



24-month warranty


We offer 24-month warranty on the product. We also provide after sales service.



Standard equipment



Important measurements

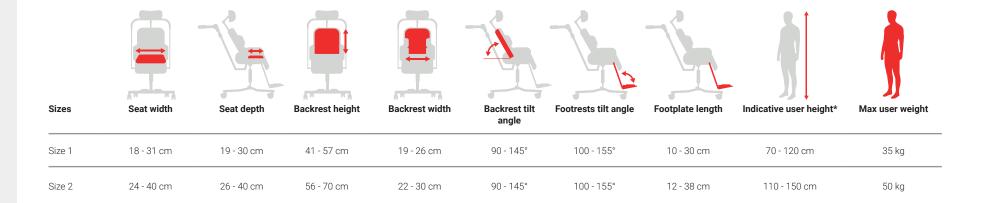

Weight and dimensions

Weight and dimensions of the Hi-Low frame

Size	Width	Length	Height (maximal)	Weight
Size 1	55 cm	97 cm	100 cm	19,5 kg
Size 2	55 cm	97 cm	100 cm	24 kg

Size	Width	Length (ngth Height (without push handles)	
Universal	55 cm	78 cm	41 - 52 cm	12 kg



Scan the QR code to see the full table.

Note: Measurement tolerance range is +/- 1 cm / ½ in.

Note: Approximate values, depending on the product configuration.

^{*}Height cannot be the main parameter while choosing the product size.

Accessories AURA® / AURA HOME®



Safety rail

Makes the user safe in the stroller. Protects against falling out.



Abduction block

Shaped abduction block provides with correct seating position. Can be easily removed.



Folding canopy

Protects against both sun and rain.



Rain cover

An excellent protection from the rain, snow, and wind either for the user and stroller.



Organizer

A handy bag attached to the push handle. Specially designed compartments and pockets make it easy to keep order and have all the necessary items at hand.



Scan the QR code, to see all the accessories.